


# Flo-Form Positioner Overview

Contaminated soft surfaces (pillows, mattresses, etc.) largely contribute to the spread of hospital-acquired infections (HAIs), threatening the safety of patients and caregivers. Soft surfaces with PneumaPure® filter technology provide a reliable barrier against microbial contamination, enhancing safety in healthcare environments.

## Flo-Form positioner features

- Patented PneumaPure® filter technology allows airflow while blocking pathogens such as MRSA, C. diff, E. coli, coronavirus, and more.
- The hermetically sealed, waterproof exterior prevents pathogen ingress and makes soft surfaces easy to clean, with no laundry needed.
- The Flo-Form® Positioner is a non-powered fluidized medium that maximizes contouring and support, redistributes pressure across a larger surface area, and promotes effective patient positioning with neutral spine and limb alignment.

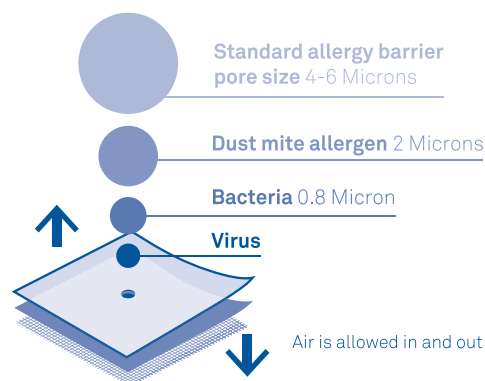

PneumaPure® Filter technology

# SleepAngel®



## Addressing the following concerns

Infection Control

Safe Patient Handling

Pressure Injury Prevention

Patient Turning & Repositioning

Patient Proning

Microclimate Compatibility

“

*Patient positioning involves properly maintaining a patient's neutral body alignment by preventing hyperextension and extreme lateral rotation to prevent complications of immobility and injury.*

# Flo-Form Positioner Features

SleepAngel®


Fluidized positioning devices (FPDs) are comprised of lubricated microspheres, giving them the ability to three-dimensionally contour to the patient's specific shape – and hold that shape until remolded. FPDs can be used for both adult and neonatal positioning.

- Flo-Form® is a non-powered fluidized medium, designed to maximize contouring and support
- The Flo-Form positioner effectively contours when normal body forces are applied
- Redistributes pressure over a greater surface area
- Effectively positions patients and promotes neutral spine and limb alignment
- PneumaPure® filter technology and waterproof exterior allow the positioner to breathe while blocking pathogen ingress and egress

Flo-Form is an excellent medium for patient positioning and pressure redistribution.



Head Positioning

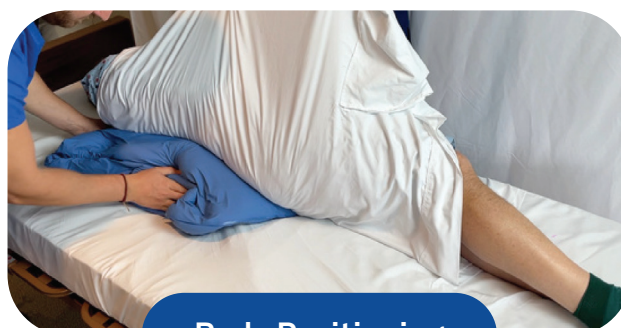

Body Positioning



Developmental Positioning

## Inspection and cleaning

Disinfect by wiping the casing with a hospital-approved disinfectant. Allow to dry. After disinfection, wipe down using a clean cloth moistened with clean water. Allow positioner to dry fully before use.

See more about detailed cleaning instructions:



# Flo-Form Positioners for Adults

SleepAngel®

Flo-Form is a fluidized technology, which can be molded to any shape and will hold its form until it is remolded. Because of this unique characteristic, Flo-Form is an excellent medium for patient positioning and pressure redistribution.



## For Head Positioning

- Supports the neck and shoulders to offload the occiput
- Protects the ears, nose, and face from shear and friction
- Supports the cervical spine
- Promotes neutral spine alignment
- Facilitates tube/line management



## For Body Positioning

- Contours and supports the body to offload the scapula and sacrum
- Maintains therapeutic positioning
- Promotes neutral body alignment
- Facilitates tube/line management



# Flo-Form NICU Positioners

SleepAngel®

Flo-Form® technology allows our neonatal positioners to be micro-contoured for each patient's unique medical needs. Flo-Form also helps to protect the skin, promote neutral body alignment, and encourage healthier development of fragile neonatal patients.



## For Head Positioning

- Redistributes pressure to protect the skin
- Helps reduce lateral skull flattening
- Promotes neutral head/trunk alignment
- Facilitates tube/line management
- Recommended for single patient use



## For Body Positioning

- Creates a soft outer boundary to support a flexed, midline position
- Helps maintain neutral body alignment and positive airway
- Encourages normal musculoskeletal and neuromotor development
- Recommended for single patient use

# Flo-Form NICU Tube Combo

SleepAngel®



## For Supination

Mold utility positioner around the outer perimeter of the baby's body, being sure to avoid any lines and tubing.



## For Proning

Build prone roll from the hips to the top of the head. Compress utility positioner below umbilicus & knees and mold larger utility positioner to the baby. Mold utility positioner around the outer perimeter of the baby's body, being sure to avoid any lines and tubing.



Begin by flattening the utility positioner.



Position baby on utility positioner and press outer edges of the positioner to mold it along the baby's perimeter.



Wrap fluidized tube around utility positioner, going under any lines or tubing. Use velcro strap to secure both ends of the tube.



Squeeze fluidized tube to flatten and build maximum height around the perimeter.



Achieve further customization by molding to the desired position.



Secure with additional tether straps where appropriate.

# Flo-Form NICU Full Body

SleepAngel®



## For Side-Lying

Mold Flo-Form against the outer perimeter of the baby's body, being sure to avoid any lines and tubing.



## For Proning

Build prone roll from hips to top of head by pushing down with fingers along side of body. Compress positioner below umbilicus and knees. A small utility positioner can be used directly on top of full body positioner as a prone roll.



Begin by flattening the Flo-Form fluidized positioner.



Position baby near top end of the positioner and mold Flo-Form toward the baby.



With "Mothers Hand" on the baby, use opposite hand to mold the far side of the positioner toward the baby.



Repeat Step 3, molding the near side of the positioner toward the baby.



Mold top of positioner to head against hand.



Mold bottom of positioner to feet against hand and place tether straps where appropriate.

# Flo-Lock Product Range

**SleepAngel®**

Name	SKU	Technical name	Size (cm)	Weight (g)	Filling
<b>Flo-Form S</b>	3015006	Flo-Form positioner, Small	19x27	312	Flo-Form
<b>Flo-Form M</b>	3015007	Flo-Form positioner, Medium	24x38	710	Flo-Form
<b>Flo-Form L</b>	3015008	Flo-Form positioner, Large	48x63	3400	Flo-Form



# Effective pressure injury prevention

Fluidized positioning devices three-dimensionally contour to the patient's specific shape – and hold that shape until remolded.

## SleepAngel®



Flo-Form can be molded to any shape and will hold its form until it is remolded.

Flo-Form is an excellent medium for patient positioning and pressure redistribution. The positioner includes PneumaPure® filter technology to prevent cross-contamination.

### Flo-Form product benefits

- Comprehensive Care: Supports pressure injury prevention, infection control, and patient turning & repositioning, ensuring safer and more effective care.
- Versatile Applications: Suitable for adult and neonatal positioning
- Sustainable Maintenance: Enables efficient cleaning and disinfection protocols, promoting hygiene while reducing resource use.
- Enhanced Efficiency: Helps reduce patient hospital days by improving outcomes and streamlining patient care processes.

### Flo-Form for adult and neonatal positioning:

- Head positioning
- Body positioning
- Developmental positioning

### For turning & side lying

- Improves caregiver safety
- Maintains therapeutic positioning
- Allows for pressure redistribution
- Promotes neutral body alignment
- Allows for customized turning and positioning



### Contact

SleepAngel Medical is produced by Gabriel Scientific

 [www.sleepangel-medical.com](http://www.sleepangel-medical.com)

 [info@sleepangel-medical.com](mailto:info@sleepangel-medical.com)